



EpiLaze

Hair Removal Parameters
Alexandrite 755nm

These treatment parameters can be used on Fitzpatrick skin types I-IV

1. Topical anesthetics may be appropriate for sensitive areas
2. Always wear appropriate and protective eye wear
3. The above settings have been used at other facilities. The treating technician is responsible for the settings that he or she decides to use on their patient.

Standard Hair Removal				
Fitzpatrick Skin Type	Description	Start Fluency (TEST proof)	Tolerated Fluencies	Pulse Width
I	Very white skin. Always burns, does not tan	9 J/cm ² -Wait at least 15-30 minutes. Observe the skin's response. Increase or decrease from 2-3 J/cm ²	9-12 J/cm ²	50ms
II	Very white skin. Always burns, sometimes tans.	12 J/cm ² -Wait at least 15-30 minutes. Observe the skin's response. Increase or decrease from 2-3 J/cm ² .	12-15 J/cm ²	60ms
III	Darker white skin. Sometimes burns, always tans.	15 J/cm ² -Wait at least 15-30 minutes. Observe the skin's response. Increase or decrease from 2-3 J/cm ² .	15-18 J/cm ²	75ms
IV-V-VI	NOT RECOMMENDED FOR SKIN TYPES IV, V & VI			
FAST Hair Removal				
Fitzpatrick Skin Type	Description	Start Fluency (TEST proof)	Tolerated Fluencies	Pulse Width
I	Very white skin. Always burns, does not tan	5 J/cm ² -Wait at least 15-30 minutes. Observe the skin's response. Increase or decrease from 2-3 J/cm ²	5-12 J/cm ²	
II	Very white skin. Always burns, sometimes tans.	5 J/cm ² -Wait at least 15-30 minutes. Observe the skin's response. Increase or decrease from 2-3 J/cm ² .	5-12 J/cm ²	
III	Darker white skin. Sometimes burns, always tans.	5 J/cm ² -Wait at least 15-30 minutes. Observe the skin's response. Increase or decrease from 2-3 J/cm ² .	5-12 J/cm ²	
IV-V-VI	NOT RECOMMENDED FOR SKIN TYPES IV, V & VI			